

## Microgen® *E. coli* O157 latex test

A rapid test for the identification and confirmation of *E. coli* serogroup O157 grown on selective solid media.

*E. coli* O157, one of a group of *E. coli* serotypes known as EHEC are the causative agents of Haemorrhagic colitis (HC), Haemolytic Uraemic Syndrome (HUS) and community acquired diarrhoea.

Sorbitol MacConkey medium is the recommended screening agar for *E. coli* O157, as the strain is unable to ferment sorbitol. However, other sorbitol-negative Gram-negative rods also grow on sorbitol MacConkey agar. As a result of this, a specific rapid confirmation method is required.

The conventional method for confirmation of *E. coli* O157 serotypes utilises specific antisera in a standard agglutination test. However, this



method and other commercially available latex tests have been shown to suffer from cross reactions with other enteric organisms such as *E. hermannii*, which antigenically and biochemically resembles *E. coli* O157, resulting in false positive results.

Rapid identification confirmed  
within 2 minutes

Easy to use

High sensitivity and specificity

# Microgen® *E. coli* O157 latex test

Microgen® *E. coli* O157 is an easy-to-use one step, latex kit which gives results in under two minutes and specifically confirms *E. coli* O157 directly from the selective agar, even in mixed culture.

Microgen® *E. coli* O157 demonstrated no cross reactions when tested against a panel of 161 *E. coli* 'O' serotypes and 56 'H' serotypes, and shows strong specific agglutination.

Microgen® *E. coli* O157 gives strong and specific results within two minutes and comes complete with a control latex which can be easily compared with test results to confirm non-specific reactions.

## Ordering Information:

Code	Description	Size	Storage
M44CE	Microgen® <i>E. coli</i> O157	50 tests	2 - 8°C

## Reagents/Kit:

Reagents	Product Code	Detects	Quantity/Size
<i>E. coli</i> O157 latex reagent	M44a	Latex particles coated with rabbit antibodies to <i>E. coli</i> O157	2.5ml
Control latex reagent	M44b	Latex particles coated with rabbit antibodies non-reactive to <i>E. coli</i> O157	2.5ml
0.85% Isotonic saline	M40	Sample preparation solution	5.0ml

If you need any further information on any of the above tests please contact Microgen Bioproducts Ltd.

**MICROGEN**  
BIOPRODUCTS

**Microgen Bioproducts Ltd**, 1 Admiralty Way, Camberley, Surrey GU15 3DT, United Kingdom  
T: +44 (0)1276 600 081 F: +44 (0)1276 600 151 E: sales.camberley@lab21.com  
[www.microgenbioproducts.com](http://www.microgenbioproducts.com)