



MICROGEN BIOPRODUCTS LTD

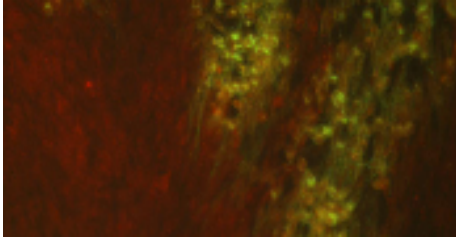

Herpes Simplex Virus Type 1

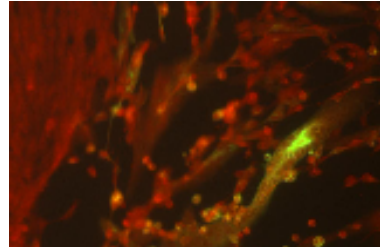

Herpes Simplex Virus Type 12

HERPES SIMPLEX VIRUS TESTING SYSTEM

Combine the D³ DFA HSV Identification or Identification and typing kit with H & V MixedCells™, flocked swabs and UTM for a complete approach to your Herpes Simplex Virus testing needs.

Protecting Food and Health

MICROGEN BIOPRODUCTS LTD

1 Admiralty Way
Camberley
Surrey
GU15 3DT
United Kingdom

Phone: +44 (0)1276 600081
Fax: +44 (0)1276 600151
E-mail: sales@microgenbioproducts.com
www.microgenbioproducts.com



Microgen Bioproducts is able to offer sensitive, specific and rapid detection of Herpes Simplex Virus 1 & 2 through cell lines of different formats and two Direct Immunofluorescence Assay's; one for the identification of Herpes Simplex virus 1 & 2 and another DFA for the identification and typing of Herpes Simplex Virus 1 and 2.

Combine the D³ DFA HSV Identification kit or the D³ DFA HSV Identification and Typing kit with Hs27 ReadyCells for culture and typing of Herpes Simplex within 24 hours. ReadyCells are stored at -70°C and can be thawed and inoculated within 5 minutes!! ReadyCells have identical performance to fresh prepared cells but without the time, cost and inconvenience of preparing fresh cells. Also, FreshFrozenCells A549, MRC-5, are available as is H&V-Mix. H&V-Mix is a mixed cell monolayer of MRC-5 and CV-1, allowing the rapid culture of HSV, CMV and VZV from lesions and other body sites in a single vial, saving time and expense.

Rapid Turnaround Time

Most lesion or genital herpes cultures show positive results within 24 hours on H&V-Mix MixedCells. Confirm CPE in as little as one 15 minute staining step.

Flexible Reagent Formats

Order as a complete kit containing reagents, control slides, PBS and mounting fluid, or order reagents independently as an identification and Typing Duo Pack (HSV-1 and HSV-2 Reagents packaged together) or Single HSV identification reagent.

Product Codes	Description
01-080000	D ³ DFA Herpes Simplex ID Kit
01-090000	D ³ DFA Herpes Simplex ID and Typing Kit
01-085005	Individual HSV ID Reagent
01-090001	HSV-1 & HSV-2 Reagents Duo Pack
01-00100	HSV-1/HSV-2 Antigen Control Slides—10 pack
98-0102	Shell Vials with Coverslip
10-32010	ReFeed Medium.
F-87-0102-12(24)	Hs27 ReadyCells (12 or 24 shell vials)
56-00050	A549 FreshFrozenCells
51-00050	MRC-5 FreshFrozenCells
98-0102	H&V-Mix