

MICROGEN BIOPRODUCTS LTD

Now with Improved Monoclonal Antibodies



D³ ULTRA

DETECT SEVEN RESPIRATORY VIRUSES IN 24 HOURS FROM R-MIX CELL CULTURES OR DIRECT FROM PATIENT SPECIMENS IN FIFTEEN MINUTES

Features

DFA Monoclonal Antibody Reagents

15 Minute Incubation

Bright Definition of Infected Cells

Liquid PBS/Detergent Concentrate

Evans Blue Counterstaining

Benefits

Making Staining a one-step process

Improves laboratory time to result

With the use of FITC labelling improves recognition of specific viral antigens

Eliminates time needed to dissolve crystalline materials

Minimises microscopist fatigue when dealing with negative samples

Protecting Food and Health

MICROGEN BIOPRODUCTS LTD

1 Admiralty Way
Camberley
Surrey
GU15 3DT
United Kingdom

Phone: +44 1276 600081

Fax: +44 1276 600151

E-mail: sales@microgenbioproducts.com

www.microgenbioproducts.com



Patients presenting with “flu like symptoms” may have viral infections other than influenza. DHI D3 Ultra DFA Respiratory Virus Screening & ID Kit can rule in or rule out all seven major pathogens or identify any one of seven viruses using highly sensitive and specific reagents in a single 15 minute incubation step.

“Identify seven major respiratory viruses in 15 minutes” from direct patient specimen materials or in incubated tissue cultures. Results of early detection (less than 48 hours from collection to result) are essential for specific patient therapy decisions.”

Quick, easy and accurate:

- For smears and scraped cells, add one to two drops. For shell vial monolayers, add 3-4 drops
- Incubate at 37 °C for 15 minutes in a humified chamber
- Wash stained cells diluted with PBS/detergent solution.
- Add mounting fluid
- Examine microscopically at 200X or higher magnification FITC filter system.

Description and Ordering Details

PART NO.	DESCRIPTION	NOTES
01-01000.	ULTRA DFA RESPIRATORY VIRUS SCREENING/ID KIT	10ML, COMPLETE KIT
01-010001.	ULTRA DFA SCREENING REAGENT SET	DIRECT SPECIMEN
01-010002.	ULTRA DFA SCREENING REAGENT SET	SHELL VIAL
01-010003.	ULTRA RSV DFA SCREENING REAGENT SET	
01-010004.	ULTRA INFLUENZA A/B DFA SCREENING REAGENT SET	1/EA X 2ML MABS
01.010005.	ULTRA INFLUENZA A/B SCREENING SET	5 EA X 2ML MABS
01-010006.	ULTRA PARAINFLUENZA DFA SCREENING REAGENT SET	
01-013010.	RESPIRATORY VIRUS DFA SCREENING REAGENT	10ML
01-013102.	INFLUENZA A DFA REAGENT	2ML
01-013202.	INFLUENZA B DFA REAGENT	2ML
01-013302.	RSV DFA REAGENT	2ML
01-013402.	ADENOVIRUS DFA REAGENT	2ML
01-013502.	PARAINFLUENZA 1 DFA REAGENT	2ML
01-013602.	PARAINFLUENZA 2 DFA REAGENT	2ML
01-013702.	PARAINFLUENZA 3 DFA REAGENT	2ML
01-014005	RESPIRATORY VIRUS ANTIGEN CONTROL SLIDES	7 VIRUSES/5 SLIDE SET