


MICROGEN BIOPRODUCTS LTD

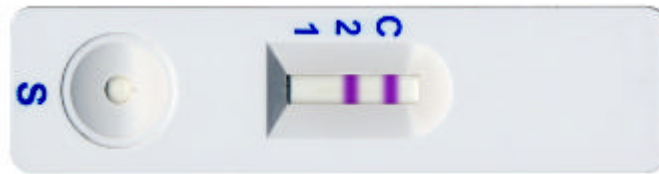

NOROSCREEN

A rapid lateral flow immunoassay for the detection of *Norovirus* antigen in faeces

Noroscreen Rapid Cassette detects and differentiates *Norovirus* Genogroup 1 and Genogroup II from faecal samples:

- **Simply:** cassette format and one-step procedure (no need to add additional conjugates or substrates).
- **Rapidly:** results within 15 minutes
- **Controls:** each cassette comes with a built in control line.
- **Ease of Reading:** clear distinct reaction lines with a clean background
- **Reliably:** high sensitivity (95.65%) and specificity (91.67%)

Protecting Food and Health

MICROGEN BIOPRODUCTS LTD

1 Admiralty way,
Camberley,
Surrey
GU15 3DT

Phone: +44 (0)1276 600081
Fax: +44 (0)1276 600151
www.microgenbioproducts.com



Noroscreen is a one-step rapid cassette which detects and differentiates *Norovirus* Genogroup 1 and Genogroup II antigen in faecal samples. Noroscreen complements the rapid Rota/adenoscreen cassette for the detection and differentiation of *Rotavirus* and *Adenovirus* in faecal specimens, allowing a laboratory to rapidly test for the 3 leading causes of viral gastroenteritis, *Norovirus*, *Rotavirus* and *Adenovirus*, with highly specific and sensitive, easy-to-use cassettes.

Noroscreen is an easy-to-use cassette with a one-step procedure which does not require the addition of conjugates or substrates:

1. Remove sample probe from preparation device and coat liberally with sample. Replace probe in vial and shake to dispense solid material.
2. Allow mixture to stand for 3 minutes.
3. Snap top from preparation device and invert. Add 2 drops to sample well of test cassette.
4. Leave the test at room temperature and read the results within 15 minutes.



Positive for Genogroup 1



Positive for Genogroup II

Product Code	Description	Kit Size
M586CE	Noroscreen cassette	10
M585CE	Rota/Adenoscreen cassette	25